

Technical Bulletin

Product:	2WIN-S – Kaleidos APP
Date:	2024/08/30 (August 30 th , 2024)
Bulletin number and title:	37 - New Kaleidos APP release 1.4.1.0
Issue has a relevance for the device safety:	NO
Field Safety Corrective Action:	Not required
Issue has a relevance for the device performance:	YES
Reason:	New Kaleidos APP software version
S/N affected:	NA
Scope of notice:	All Distributors/Other
Service time:	Less than 1 minutes for software update procedure
Service parts:	Google Play Store (App) Installation file: <i>Kaleidos_1.4.1.0_(58)_release.apk</i>
Description:	See next page/s
Required action:	Update recommended
Hardware compatibility:	2WIN: from BOM 29.1; compatible tablets cited in the Operator Manual
Software compatibility:	2WIN: from 5.4.2; tablet: Android 8, 9, 10, 11, 12, 13
For further info please contact:	<p>Adaptica Srl Via San Marco, 9/h – 35129, Padova (PD), Italy Tel: +39 049 773 968 Fax: +39 049 097 0901 Email: support@adaptica.com</p>

Summary of changes and bugs fixed in Kaleidos APP SW version 1.4.1.0

- Added possibility to change the interface language
- Added Screening Table's result to the exported CSV file
- Added filter option to easily select patient on Screening Mode
- Added possibility to save a CSV file with measurement results also in normal mode
- Added possibility to customize Screening Table's thresholds (through 2WIN), whose default values have been updated to AAPOS 2021